


ICV

Product Models and Versions Covered in This Document



Models	Versions	Description
Preference	A	Dental Furniture

Fill the ICV Tank


Fill the tank when the low fluid indicator is lighted.

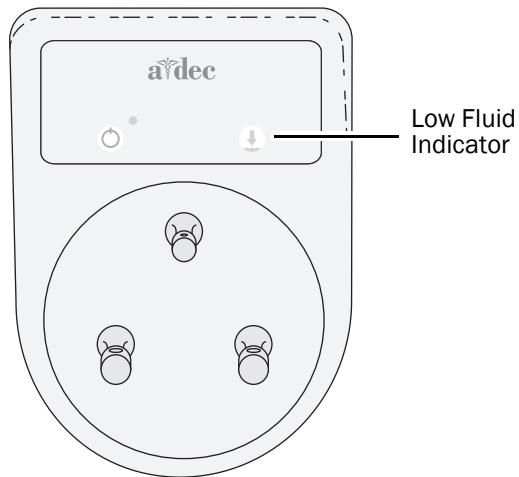

1. Pull the blue water tubing and the water level connector off of the tank's connectors.

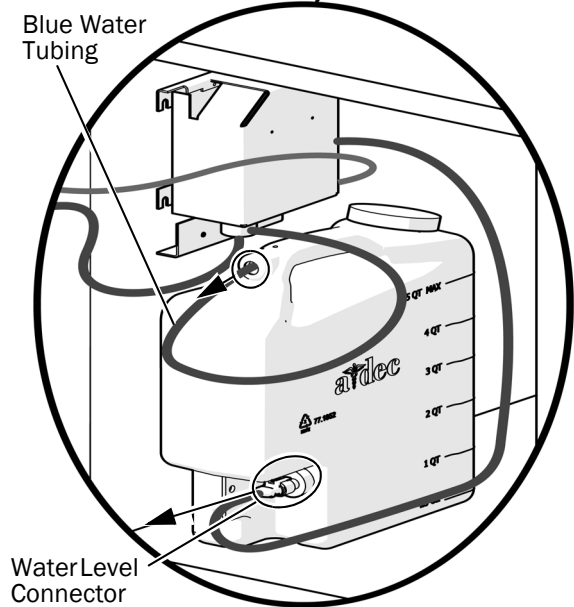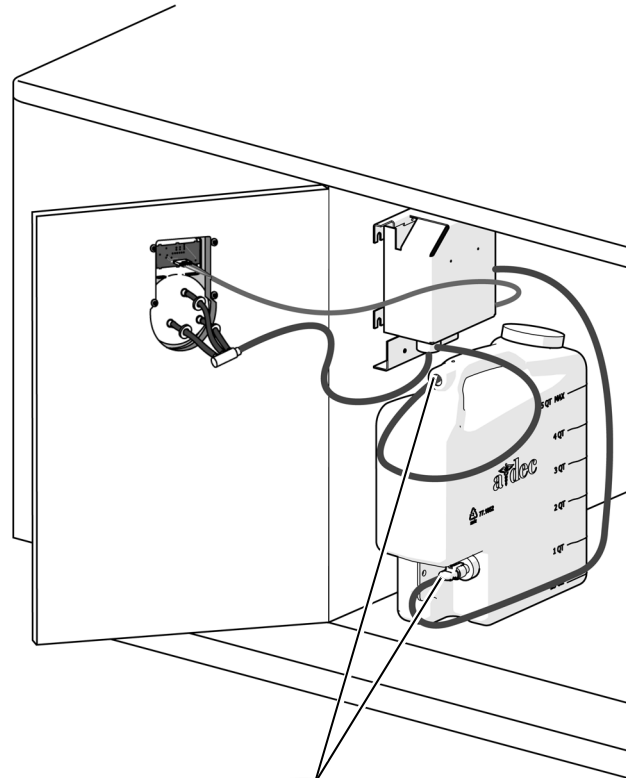

2. Remove the cap from the tank and add cleaner and water per the cleaner manufacturer's recommendation.

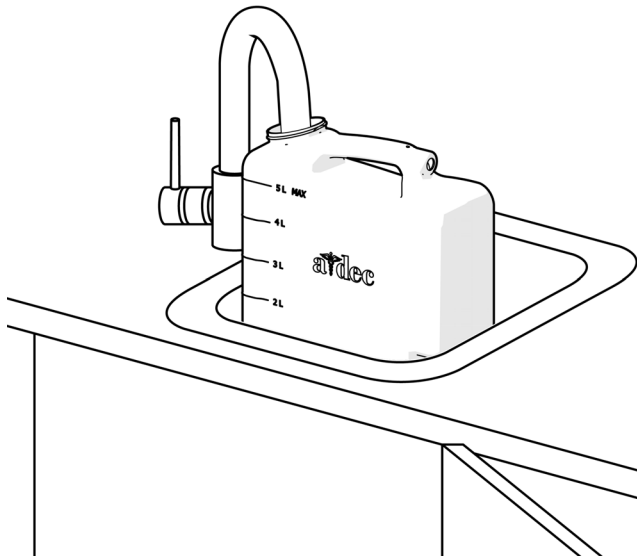

3. Reinstall the cap on the tank.
4. Reconnect the water tubing and the water level connector on the tank's connectors.

Clean the Vacuum Instrumentation




CAUTION Before you press the start/stop button, ensure that at least one tubing is attached and the tubing valve is open. This prevents damage to the cabinet face.

1. Connect the handpiece tubing to the ICV.
2. Open the valve(s).
3. Press the start/stop button.



NOTE This product is rated 100-120/220-240 VAC, 50-60 Hz, 0.5 A.

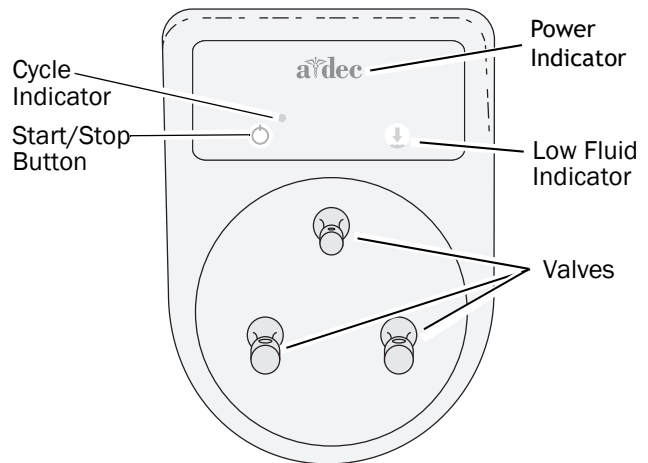

Regulatory Information and Warranty

For required regulatory information and the A-dec warranty, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00) available in the Resource Center at www.a-dec.com.

A-dec Headquarters

2601 Crestview Drive
Newberg, OR 97132 USA
Tel: 1.800.547.1883 Within USA/CAN
Tel: +1.503.538.7478 Outside USA/CAN
www.a-dec.com

A-dec, Inc. makes no warranty of any kind with regard to the content in this document including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

